

Durex Flavours 6 Condoms



Mixed flavors for a little sweetness Mixed scents to delight your senses Sugar free Easy-on shaped & teat ended for comfort and fit Colored and lubricated natural rubber latex condoms Nominal width: 56mm 100% electronically tested

Rating: Not Rated Yet

Price

USD16.00

USD16.00

[Ask a question about this product](#)

Manufacturer [Durex](#)

Description

Durex Flavours Condoms is a selection of fun fruit flavours: Banana, strawberry, orange and apple flavour bring loads more fun and also an added dash of extra excitement into sex.

Durex Flavours Condoms are indeed the condoms you need to spice things up with your partner.

- Use a new condom for each act of sex.
- Tear open the package carefully. Do not use fingernails, teeth, or anything that can damage the condom.
- Before any sexual contact, place the condom on the head of the erect penis with the rolled side out.
- Unroll or pull the condom all the way to the base of the erect penis. If the condom doesn't unroll easily, it may be on backwards, damaged, or too old. Throw it away and start over with a new condom.
- Immediately after ejaculation, hold the rim of the condom in place and withdraw the penis while it is still erect.
- Dispose of a used condom by wrapping it in tissue and throwing it into the trash. Wash your hands with soap and water.

Reviews

There are yet no reviews for this product.