

Medihoney Antibacterial Medical Honey 20g



Wound gel

Rating: Not Rated Yet

Price

USD19.83

USD19.83

[Ask a question about this product](#)

Manufacturer [Derma Sciences](#)

Description

Description:

Medihoney 100% Pure Medical Grade Manuka Honey in a single patient sealable, reusable tube.

Indications:

- Deep wounds and sinus cavities
- Infected and malodorous wounds
- Necrotic and sloughy wounds
- Surgical, post op wounds, donor sites and recipient graft sites
- Leg ulcers (venous, arterial and mixed aetiology ulcers) and diabetic foot ulcers
- 1st and 2nd degree burns

Features:

- A totally natural product which is 100% pure medical grade Manuka honey
- Provides broad spectrum antimicrobial activity and is effective against biofilms due to MGO content being above 355mg/Kg
- Is anti-inflammatory
- Eliminates malodour, thus improving patient's quality of life
- Provides autolytic debridement through the osmotic action, effectively removing necrotic tissue and wound slough
- Adjusts wound pH to the range 3-4 creating the optimum environment for wound healing
- Is safe to be used on all ages including paediatrics and neonates
- The sealable tube can be used for up to 4 months from opening
- Extensive clinical evidence across a large range of wound types and skin conditions

Use:

Protect wound edges with Medihoney Barrier Cream.

Apply the honey directly to the base of the wound, avoiding direct contact between the wound and the nozzle of the tube.

The honey can also be applied into deep wound cavities or soaked into a suitable dressing.

Ensure that the Medihoney is in full contact with the base of the wound - approximately 3mm thickness (1/4 inch).

Apply more if wound fluid is present.

A suitable non-adherent sterile pad or dressing should be applied to cover the honey.

The dressing should be sufficiently absorbent to manage wound fluid.

Medihoney Antibacterial Medical Honey can be left in the wound for up to 7 days dependent on the amount of fluid present.

The dressing should be changed when it is saturated with wound fluid or if the cover dressing is leaking.

Reviews

There are yet no reviews for this product.