

## Thera Pearl Neck Wrap



Reusable Hot & Cold Therapy

Rating: Not Rated Yet

**Price**

USD21.00

USD23.33

[Ask a question about this product](#)

Manufacturer: [THERA°PEARL](#)

### Description

**Description:**

Thera Pearl Neck Wrap ease the tension in your neck and shoulders.

Poor posture and overwork can leave your neck and shoulders feeling tight and sore. Instead of an expensive massage, drape THERA°PEARL's flexible Neck Wrap over your aching muscles. You won't believe how quickly its comforting hot or cold therapy breaks the tension.

The wrap's wide back and long sides blanket your neck and shoulders simultaneously for twice the relief. Twist it into any position even when it's frozen! It's so lightweight and comfortable you can wear it under a shirt or sweater.

Chill it in the fridge or freezer, or pop it in the microwave. The wrap holds its therapeutic temperature for a full 20 minutes. Numb the ache with analgesic cold, or soothe and release the tightness with penetrating heat. Your neck and shoulders will thank you.

Neck Wrap is Perfect for:

- Stiff Neck
- Sore Shoulders
- Spine Pain & More!

### Directions

THERA°PEARL Pack should be applied to skin for doctor recommended time of 20 minutes. Remove and wait at least 20 minutes. Reapply pack for 20 minutes, if needed.

**HEAT THERAPY** - Microwave for time indicated below and test pack temperature. Heat for additional 5 seconds as needed until desired temperature is reached.

## Thera Pearl Neck Wrap

---

- 700 Watt Microwave: 60 seconds
- 1000 Watt Microwave: 50 seconds
- 1250 Watt Microwave: 40 seconds

**COLD THERAPY** - Freeze for at least 2 hours.

### Reviews

There are yet no reviews for this product.