

Urgo Ampoules 5 large dressings



Heel blisters treatment

Rating: Not Rated Yet

Price

USD11.00

USD11.00

[Ask a question about this product](#)

Manufacturer [Urgo](#)

Description

Description:

Thanks to its thickened central pellet , URGO Ampoules isolates the bulb from pressure and friction to immediately relieve pain . In contact with the bulb, hydrocolloid technology forms a dermo-restorative protective gel that promotes the healing of the bulb. Ultra resistant and adapted to the shape of your heel , URGO Ampoules is the reference dressing to treat your localized bulbs in the heel.

Features:

- over-thickened center pellet
- second skin effect
- relieves immediately
- promotes healing

Use:

1. Do not pierce the bulb. If the bulb is already pierced, clean with soap and water and apply antiseptic.
2. Remove the transparent winglet (arrow zone) and warm your dressing in the palms of your hands before removing the decorated paper.
3. Apply to clean, dry skin by positioning the center of the dressing directly on the bulb.
4. Carefully smooth the edges to maximize adhesiveness.
5. The dressing will peel off after 2 to 3 days. It is advisable to wait for the dressing to come off spontaneously.

Reviews

There are yet no reviews for this product.